



Escape Chair VOLT

Advantages Escape-Chair VOLT:

- Powered support makes going up the stairs much easier for operator.
- Ideal for quick evacuation (4 speed descent).
- Lithium cell enough for up/down 40 floors.
- Single operator chair.
- Ready for use within seconds.
- Easy to use and operate (wireless).
- Equipped with arm- and footrest.
- 4 height adjustable operating handle.
- Battery indicator on display control panel.
- LED light for dark environment evacuation.
- CE certified.



Powered stair climbing evacuation & rescue chair...

What's an Escape-Chair VOLT?

An **Escape-Chair® VOLT** is the **POWERED** solution to evacuate & rescue people to safety in a seated position up and/or down a flight of stairs. A simple procedure can make the **Escape-Chair VOLT** fully functional within a few seconds.

How does the Escape-Chair VOLT work?

The **Escape-Chair VOLT** is operated by a self-explanatory control panel with display on the operating handle. After unfolding the **Escape-Chair VOLT** it is immediately ready for use. The single operator must push the operating handle down in the direction of the stairs.

Specifications	Escape-Chair VOLT
Article number:	10050
Dimensions folded up (H x W x D):	109 x 63 x 29 cm
Maximum load bearing:	254 kg
Maximum stair gradient:	40°
Weight:	±37,5 kg
Using time when full charged:	2 hours (±40 floors)
Type of tracks:	Serrated tracks for superior grip
Unlocking function tracks:	✓
Battery life:	2 – 3 years
Extra battery pack:	Optional
Number of speeds:	4
Ready for use within seconds:	✓
Foot rest & Arm rests:	✓
LED lightning for dark environment:	✓
4 height adjustable operating handle:	✓
Flame & heat resistant dust cover:	✓
Carrying handles front & back:	✓
10G crash test bracket for Ambulances	✓
CE certified:	✓
Wall mounting bracket:	✓
Delivery included:	✓
Warranty on non-wearing parts	10 years

Our advice:

Make sure you have an **Escape-Chair VOLT** available in your organization so that you're prepared for any evacuation and/or emergency, for up and/or down evacuation.

"Failure to prepare for an emergency has its consequences."



Escape Mobility USA.

1261 S Lyon Street Suite 405 Santa Ana, CA 92705, USA

T +1 (855) 470-4410

W www.escape-mobility-usa.com

